EN **USER MANUAL**
FINGERTIP PULSE OXIMETER

ITA **MANUALE D'USO**
SATURIMETRO DA DITO

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HC-FTPOY2

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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REF

YM101

Shenzhen Yimi Life Technology Co.,Ltd.
305, Tengdo Industrial Park, Changshangjiang Street,
Londbei Village, Pingshan District, Shenzhen, 518118, China

EC REP

Share Info Consultant Service LLC Repräsentanzbüro.
Heerdter Lohweg 83, 40549 Düsseldorf, Germany

Distributed by / Distribuito da:
Mach Power Italy s.r.l.: Via Alberigo Albricci, 8 - 20122
Milano (MI) Italy

RX ONLY

CE 0123

Product Introduction & Operation Guide

1.1 Front View

1.2 Operation Method

A. Open the battery cover, and put the two AAA batteries into the battery compartment in correct polarities, then replace the cover;

B. Press the bottom of the equipment and open the probe, then insert one finger into the probe;

C. Press the button to turn the equipment on, and the measure interface will appear;

D. After about 8 seconds, the measurement result can be read directly from the display screen;

E. Before reading the parameters, make sure that stable numbers of the pulse oximeter interface has sustained more than 4 second;

F. The equipment will turned off automatically within 8 seconds when the finger left the probe.

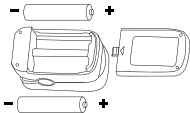
1.3 Battery Installation

A. Put the two AAA batteries into battery compartment in correct polarities.

B. Push the battery cover horizontally along the arrow shown as right.


WARNINGS:

- Battery polarities should be correctly installed, otherwise, damage may be caused to the equipment.
- Please remove the batteries if the equipment will not use for a long time.

1.4 Lanyard installation

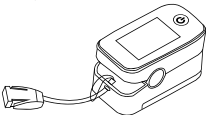- Pass the thinner end of the lanyard through the hanging hole.
- Pass the thicker end of the lanyard through the thinner end and tighten the lanyard.

1.5 Attention for Operation

- Before use check and confirm that the people or finger size were applicable;
- Before use check and confirm that the environment should be non-combustible material, as well as to avoid

high or low temperature and humidity, but also need to pay attention to the following:

a) To avoid glare and direct sunlight exposure;

b) To avoid radiation infrared or ultraviolet radiation;

c) Avoid contact with the organic solvent, mist, dust, corrosive gases;

C. The equipment should not be used at a location or limb tied with arterial canal or blood pressure cuff or receiving intravenous injection;

D. The equipment may not work normally on microcirculation barrier patients, Warm or rub the finger, or re-position the equipment could improve the measurement.

E. The ray between photo detector and light emitting diode should cross patient's arteriole.

F. The patient should not use enamel or other makeup;

G. Avoid to insert a wet finger into the probe.

Notes:

A. The user should fully insert the finger into the probe,

B. It is recommended to let the LED light shine directly on the nail;

C. Don't shake the finger and try to keep the patient still during the measurement.

1.6 Functions and Menu Operation

1.6.1 The button operation rules

Long-press functions include entering menus, activating item's submenu, confirming setting values, and exiting item's submenu; short-press functions are polling menu items and viewing the setting values of items. It should be noted that long-press means pressing the key

Should be noted that long-press means pressing the key for about 2 seconds, and short-press means pressing the key for less than 0.5 second.

1.6.2 Switch display direction

Short-press to switch the display direction

1.6.3 Menu Operation

Active the menu

After the oximeter is turned on, long-press the power button to activate the menu, then short-press the button to view the setting values of each item. If the user wants to change the setting value of the item, long-press to enter the item's submenu, the parameter value starts to flash, short-press to traverse the parameter value until the parameter value required by the user is selected, long press to confirm and exit the submenu.

Setup the LED display brightness

The first item is to setup the display brightness. Long-press the button to select a brightness level ranging from 1 to 3. The greater the value, the greater brightness of the display.

Setup the SpO2 Alarm Limits

The second item is to setup the SpO2 alarm limits. For example: When SpO2 High limit is set to 96, an alarm will be issued when the SpO2 value is higher than 96, and when SpO2 low limit is set to 94, an alarm will be issued when the SpO2 value is lower than 94.

Setup the PR Alarm Limits

The third item is to setup the PR alarm limits. For example: When PR High limit is set to 130, an alarm will be issued when the PR value is higher than 130, and when PR low limit is set to 50, an alarm will be issued when the PR value is lower than 50.

Turn Sound On/Off

The forth item is long-press to turn sound on /off.

Check the software version

The fifth item is to view the software version.

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Mach Power® is a registered trademark